

With COVID cases back on the rise, Logistics will be rolling out a new cleaner (Quick Defense NDC) that is EPA registered for use against SARS-COV-2 (COVID-19). This cleaner will be phased in as the inventory of wipes run out.

***What exactly is Quick Defense replacing?***

- All wipes (red top, purple top, Clorox, and peroxide)
- Bleach 1:10 solution on units (we will still have bleach available at P100 for decontamination of any CDIF positive transports)
- Simple Green

***How do I use?***

- When handling items soiled with blood or body fluids, use disposable gloves, gowns, mask and eye protection
- To disinfect hard non-porous surfaces, spray 6-8 inches from surface
- Spray 6-8 inches from surface
- Let sit for 5 minutes
- Wipe with task brand wipes (pop top wipes)

***What can I use Quick Defense on?***

- Stretchers
- Floor of units
- Monitors \*\*\*you should still use alcohol spray provided for monitor cables\*\*\*
- PCR Tablets
- If you have a patient with C Diff you will need to contact CMED to return to P100 for decontamination

***When should I NOT use Quick Defense?***

- On CPR puck
- **Don't mix with bleach!!!!**

***What does Quick Defense do?***

- Cleaning: Removing visible dirt and debris from surfaces by washing or wiping with detergents
- Sanitizing: Reducing the number of bacteria on a surface to a safe level
- Disinfecting: Complete removal or inactivation of bacteria, fungi and viruses on a surface

***How do I use the refill station?***

- The Simple Green refill station has been replaced with "Quick Defense" and works the same
- Place the empty spray bottle under the refill tube
- Press the corresponding black button.
- Release button once bottle has been filled to desired level
- Ensure the bottle you are refilling has a "Quick Defense" label and is not bleach